

JetFresh

ANTI-SNORING CPAP

# FACE BAND

with Adjustable Straps

Designed with ergonomic precision, our anti-snoring chin strap offers a comfortable fit while effectively reducing snoring—helping you (and your partner) enjoy a quieter, more restful night.



JetFresh

## PRODUCT DETAILS

- ✦ **Cleaning:** Hand wash only with mild detergent
- ✦ **Material:** 76.8% Nylon, 23.2% Spandex
- ✦ **Storage:** Keep in a cool, dry place
- ✦ **Features:** Reusable and washable


## ADJUSTMENT PERIOD

- **Initial Fit:** May feel snug during first use.
- **Adapts with Use:** After 2–3 days, the strap will naturally stretch and conform to your face.
- **Comfort Grows:** With continued use, you'll notice less tightness and more comfort.

## DIRECTIONS FOR USE

### STEP 01

**Unfold the chin strap** and identify the short side, which should be placed close to your body. Position your chin in the middle section of the strap.

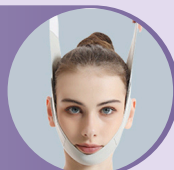

### STEP 02

**Align your ears with the holes** on both sides to ensure a comfortable fit and stick the top Velcro strap over your head



### STEP 03

**Secure the back Velcro** to adjust the tightness around the head. Ensure its snug but comfortable.



### STEP 04

**Adjust the top Velcro strap** on the head again if needed.




### STEP 05

**Fine-tune the side Velcro straps** near the ears for a perfect fit and stability.



### TIPS

Make sure the strap is snug enough to keep the mouth closed but not so tight that it's uncomfortable.

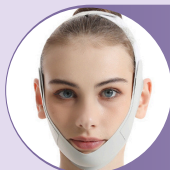

## CAUTIONS:

- ! Stop using if irritation occurs, see a doctor if necessary.
- ! Not recommended for people with TMJ (jaw disorders) without medical advice.
- ! Always consult with your sleep specialist if you experience discomfort or airflow obstruction.
- ! It is recommended to clean it once before using, to eliminate the smell from the original material.